

# URINALYSIS

## Collection Instructions for SNF/LTCF Study

### KIT COMPONENTS



RED/YELLOW  
VACUTAINER



TRANSFER  
STRAW



REQUISITION  
FORM



SPECIMEN  
BAG



FEDEX SHIPPING  
BAG AND LABEL



TRANSPARENT  
BLUE BAG


### STEP 1

Using a sterile technique, collect a minimum of 8 mL voided or catheter urine.



### STEP 2


Place transfer straw into collected urine sample.

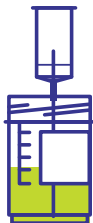

Caution: needle inside transfer straw cup.

### STEP 3

Take the Vacutainer and push the rubber top into the transfer straw. Hold in place until filled to minimum fill line. Remove Vacutainer and dispose of transfer straw in sharps container.

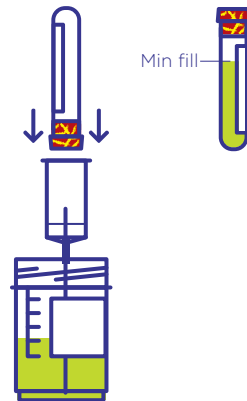

# PACKING AND SHIPPING

## STEP 4


Fill out the test requisition completely.

If ordering another test as a reflex, only one test requisition form should be completed. Package reflex test specimen(s) with this test requisition.



## STEP 5

Adhere barcode label with at least two patient identifiers on Vacutainer.

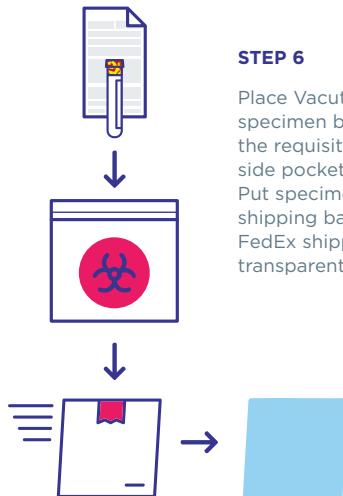

## STEP 6

Place Vacutainer into specimen bag and seal. Place the requisition form into the side pocket of specimen bag. Put specimen bag into FedEx shipping bag and seal. Place FedEx shipping bag inside transparent blue bag and seal.