


URINE CULTURE ID & SENSITIVITY

Collection Instructions for Mixed Age Study

PATHNOSTICS[®]
MOVING CARE FORWARD

KIT COMPONENTS

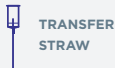

STEP 1


Using a sterile technique, collect a minimum of 4 mL voided or catheter urine.



STEP 2


Place transfer straw into collected urine sample.

Caution: needle inside transfer straw cup.

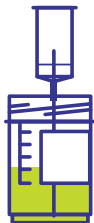

STEP 3

Take the Vacutainer and push the rubber top into the transfer straw. Hold in place until filled to minimum fill line. Remove Vacutainer and dispose of transfer straw in sharps container.

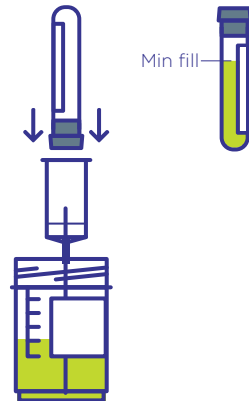

PATHNOSTICS[™]
MOVING CARE FORWARD



PACKING AND SHIPPING

STEP 4

Fill out the test requisition completely.

STEP 5

Adhere barcode label from requisition on Vacutainer. Include at least two additional patient identifiers, such as patient name and date of birth, on the label.

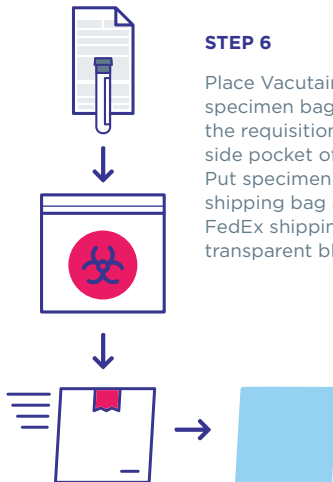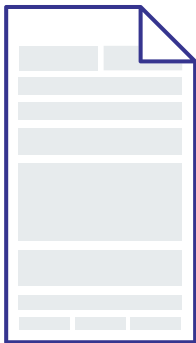

STEP 6

Place Vacutainer into specimen bag and seal. Place the requisition form into the side pocket of specimen bag. Put specimen bag into FedEx shipping bag and seal. Place FedEx shipping bag inside transparent blue bag and seal.